

AlterG® Shorts Care

The Anti-Gravity Treadmill® from AlterG can be found in hundreds of Joint Commission Certified facilities across the United States spanning nearly a decade of use. The care and cleaning procedures below have proven effective in care settings in these stringent environments.

The shorts used with the AlterG® Anti-Gravity Treadmill® are available in a wide range of waist sizes (3XS-5XL) to provide a comfortable fit for all body types. Made from high-quality neoprene and polyester, the shorts will provide excellent performance in both clinical and athletic environments.

Disinfection & Cleaning.

Shorts should be disinfected between uses to minimize the spread of germs. Saturation of shorts with a non-toxic commercial-grade disinfectant and air drying is strongly recommended. Successful disinfectant products tested include PureGreen24™ & U-Sol Surface Sanitizer™. Consult the Center for Disease Control website for the latest guidelines. Use of chlorine bleaches may void the warranty. The shorts can tolerate immersion in a 10% non-chlorine bleach solution; however, this may lead to discoloration of the shorts.

If shorts become soiled due to heavy sweating or other bodily fluids, shorts should be hand-washed, or placed on a gentle cycle with mild soap and then air dried. Never place shorts into a dryer, as this will cause deterioration of the shorts, shortening effective life.

Warranty

Shorts are warranted against manufacturer's defects for 90 days following purchase. If you receive defective shorts, please contact AlterG@golifeward.com or call +1.510.270.5900 to report the problem. If your shorts are within the warranty time frame, AlterG will send you a replacement pair at no charge with a pre-paid return shipping label. To avoid being charged, you must ship back your defective shorts to AlterG using the return shipping label.

For more information, please visit us at alterg.com or call us at +1.510.270.5900.